npuri[®]

by Bambiboo

FACTSHEET – INSTRUCTION OF USE – SAFETY INFORMATION – CONTACT DETAILS

EN – PL – DE – CZ – ES – FR – HU – IT – NL – RO – SK – SV

BRAND NAME:

npuri by Bambiboo

PRODUCT NAME:

npuri by Bambiboo selfwarming steam eye mask

KEY INGREDIENTS:

Next to your skin: chlorine-free non-woven fabric and soft elastic ear loops. Inner pocket: non-woven pouch with a self-heating insert (iron powder, activated carbon, sodium polyacrylate, sodium chloride, water, vermiculite). Fragrance-free product.

Dispose of the box in the paper recycling bin and the sachet in the plastic recycling bin.

Made in China.

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Before applying the mask, remove makeup and cleanse your face and eyelids. Take the mask out of the sachet and unfold it. Then place it over closed eyes and hook the ear loops around your ears. The mask will start to warm up after being removed from the sachet – use it immediately after opening. The maximum temperature does not exceed 45°C (average temperature: 42°C). The mask stays warm for about 20 minutes. Dispose of the used mask in a bin.

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WARNING: To avoid the risk of suffocation, keep the packaging and pads out of the reach and sight of children and pets. Single-use product.
EXPIRY DATE: see packaging.

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For home use only. Do not use the mask if it is damaged, unsealed, or past its expiration date. Contact lens or eyeglass wearers should remove them before use. Do not use eye drops immediately before applying the mask. Not for use by children. Pregnant women and people with eye or skin conditions, or those who have recently undergone facial procedures, should consult a doctor and exercise caution when applying the mask. If you experience any discomfort, burning sensation, redness, or irritation of the eyes or eyelids while using the mask, discontinue use immediately and consult a doctor if necessary.

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